



ENEMA ADMINISTRATION SIMULATOR LF00957U INSTRUCTION MANUAL



Life/form® Products by NASCO

About the simulator...

The **life/form**® Enema Administration Simulator had been designed to demonstrate and practice enema procedures. Standards apparatus can be used to introduce fluids. This simulator must be treated like a live patient or damage will result.

List of components

1. 1 Enema bag
2. 1 Latex glove
3. NASCO Hevi-Lube lubricant
4. 18" Vinyl tube
5. Carrying case

Internal Structure

All aspects of the simulator are lifelike in every detail, and the action of the unit is extremely realistic. The upper buttock is flexible so that the trainee must raise the buttock in order to locate and make into the anus. Simulated bowl will accommodate the introduction of fluids.

General Instructions for Use.

Set up procedure

1. Fill flexible Enema Administrator Unit will clear water.
2. Hang a bag a maximum of 18" above the simulator.
3. Prior to each use of the simulator, the anal orifice and enema catheter **MUST** be lubricated with NASCO Hevi-Lube Lubricant (LF01050UA). Must not use substitutions. (CAUTION: Do not use large amounts of NASCO Hevi-Lube. Over application may block drain tube.)



4. The drain tube, which is stored in the case, must be attached to the connector on top of the simulator before the introduction of fluid. Place the drain tube in a sink, or bucket (not included) for continuous drainage of administered fluids.

Procedures That Can Be Performed on This Simulator

The standard enema procedures can be performed using conventional enema apparatus, no special equipment is required.

Care of the Simulator

Normal soil can be removed with mild soap and lukewarm water. Use REN cleaner (WA09919UA) to remove stubborn stains. Spray soiled area with cleaner and wipe clean with soft cloth or paper towel.

Before storage, drain completely. Hold simulator upside down and drain all water. Remove 18" drain tubing. Dry all surfaces.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator of any kind of printed paper or plastic. These materials will transfer an indelible stain. Ball-point pens will also make a permanent stain.

Supplies/Replacement Parts for Enema Administration Simulator

W09919UA	REN Cleaner
LF01050UA	NASCO Hevi-Lube
LF01024UA	18" Drain Tubing

Other Available *Life/form* Simulators

LF00698U	Adult Injectable Arm (White)	LF03610U	Child Airway Management Trainer Head Only
LF00856U	Female Catheterization	LF03611U	Child Defibrillation Chest Skin
LF00901U	Prostate Examination	LF03612U	Child IV Arm
LF00906U	Ostomy Care	LF03613U	Child Blood Pressure Arm
LF00929U	Surgical Bandaging	LF03614U	Child Intraosseous Infusion/Femoral Access Leg Only
LF00957U	Enema Administration	LF03615U	Complete Child CRiSis™ Update Kit
LF00958U	Pediatric Injectable Arm	LF03616U	Child CRiSis™ Manikin
LF00961U	Intramuscular Injection	LF03617U	Deluxe Child CRiSis™ Manikin with Arrhythmia Tutor
LF00984U	Breast Examination	LF03620U	PALS Update Kit
LF00995U	Arterial Puncture Arm	LF03621U	Infant Airway Management Trainer Head Only
LF00997U	Adult Injectable Arm (Black)	LF03622U	Intraosseous Infusion Right Leg
LF00999U	Pediatric Injectable Head	LF03623U	Infant Airway Management Trainer
LF01008U	Intradermal Injection Arm	LF03626U	Child Femoral Access Injection Pad Replacement
LF01012U	Heart Catheterization (TPN)	LF03632U	Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF01019U	Ear Examination	LF03633U	Child Airway Management Trainer with Torso
LF01020U	Supplementary Ear Set	LF03693U	Basic Buddy CPR Manikin
LF01025U	Male Cath-Ed I	LF03699U	"Airway Larry" Airway Management Trainer
LF01026U	Female Cath-Ed II	LF03720U	Baby Buddy Infant CPR Manikin
LF01027U	Peritoneal Dialysis	LF03953U	CRiSis™ Manikin
LF01028U	Suture Practice Arm	LF03955U	Deluxe CRiSis™ Manikin
LF01036U	Spinal Injection	LF04001U	GERi™ Nursing Manikin
LF01053U	Cross-Sectional Anatomy, Torso, Head	LF04020U	KERi™ Nursing Manikin
LF01054U	Cross-Sectional Anatomy, Head	LF04021U	KERi™ Basic Manikin
LF01062U	Pelvic, Normal & Abnormal	LF04022U	KERi™ Advanced Manikin
LF01063U	Stump Bandaging, Upper	LF04030U	GERi™ Advanced Manikin
LF01064U	Stump Bandaging, Lower	LF04040U	GERi™ Basic Manikin
LF01069U	Cervical Effacement		
LF01070U	Birthing Station		
LF01082U	Cricothyrotomy		
LF01083U	Tracheostomy Care		
LF01084U	Sigmoidoscopic Examination		
LF01087U	Central Venous Cannulation		
LF01095U	Blood Pressure Arm		
LF01108U	Intraosseous Infusion Simulator		
LF01142U	Auscultation Trainer		
LF01162U	Venotech IV Trainer		
LF03000U	CPARLENE® Series		
LF03601U	Adult Airway Management Trainer		
LF03602U	Adult Airway Management on Manikin		
LF03603U	Adult Airway Management Head Only		
LF03609U	Child Airway Management Trainer		

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com