



---

**CHILD AIRWAY  
MANAGEMENT TRAINER  
LF03609U  
INSTRUCTION MANUAL**

---



***Life/form***® Products by NASCO

## About the Simulator . . .

The **Life/form**® Child Airway Management Trainer is the most realistic simulator available for the training of intubation skills. NASCO has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail.

Landmarks include: nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

The **Life/form**® Child Airway Management Trainer allows you to practice oral, digital, and nasal intubation. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

### List of Components:

1. Airway Management Trainer Head with Lungs and Stomach
2. Mounting Board
3. 8 oz. Pump Spray Lubricant
4. Case

### Lubrication:



Figure 1

Before using the **Life/form**® Child Airway Management Trainer, lubricate both the tube and airway with the NASCO lubricant provided (Figure 1).

**Please Note:** NASCO recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Airway Management Trainer. The use of a silicone or similar lubricant may cause damage to the simulator, thus voiding NASCO's warranty on the trainer.

### Cleaning the Face:

Normal soil can be removed from the face of the trainer with mild soapy water. REN cleaner (W09919U) will remove stubborn stains. Simply apply REN to the soiled area and wipe clean with a soft cloth or paper towel.

Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains. Do not use any cosmetics.

### Removing the Child Airway Management Trainer Head from the Mounting Board:

If you wish to remove the head from the board:

1. Remove the lungs and stomach from the trainer by loosening the clamps that hold them in place. The clamps are loosened by twisting each side of the clamp in opposite directions.
2. Now turn the head until the keys on the neck align with the slots in the mounting bracket and slide out (Figure 2).



## Available Supplies:

- LF03608U** NASCO Lubricant
- LF03627U** Replacement Lungs
- LF03628U** Replacement Stomach
- LF03631U** Replacement Headskin with Airway

**Figure 2**

3. To replace the head on the mounting board simply reverse the previous two steps.

## Important — Storage of Your Simulator:

It is vital that the packing foam is retained for correct storage of your simulator. The unit must be stored as you received it to avoid damaging the simulator. Follow these steps when placing the simulator in the case for storage or transport.

1. Place the board in the case so that the wide end of the board is positioned under the 2 cleats inside the case.
2. Making sure the head is in the mounting bracket, turn the head 90° to **it's** left so that it is facing the hinged side of the case.
3. Place the foam block into position over the neck, just below the chin, so that the round hole is up.
4. The case can now be closed and your manikin will be protected from excessive movement within the case.

## Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)  
**LF00856U** Female Catheterization  
**LF00901U** Prostate Examination  
**LF00906U** Ostomy Care  
**LF00929U** Surgical Bandaging  
**LF00957U** Enema Administration  
**LF00958U** Pediatric Injectable Arm  
**LF00961U** Intramuscular Injection  
**LF00984U** Breast Examination  
**LF00995U** Arterial Puncture Arm  
**LF00997U** Adult Injectable Arm (Black)  
**LF00999U** Pediatric Injectable Head  
**LF01008U** Intradermal Injection Arm  
**LF01012U** Heart Catheterization (TPN)  
**LF01019U** Ear Examination  
**LF01020U** Supplementary Ear Set  
**LF01025U** Male Cath-Ed I  
**LF01026U** Female Cath-Ed II  
**LF01027U** Peritoneal Dialysis  
**LF01028U** Suture Practice Arm  
**LF01036U** Spinal Injection  
**LF01053U** Cross-Sectional Anatomy, Torso, Head  
**LF01054U** Cross-Sectional Anatomy, Head  
**LF01062U** Pelvic, Normal & Abnormal  
**LF01063U** Stump Bandaging, Upper  
**LF01064U** Stump Bandaging, Lower  
**LF01069U** Cervical Effacement  
**LF01070U** Birthing Station  
**LF01082U** Cricothyrotomy  
**LF01083U** Tracheostomy Care  
**LF01084U** Sigmoidoscopic Examination  
**LF01087U** Central Venous Cannulation  
**LF01095U** Blood Pressure Arm  
**LF01108U** Intraosseous Infusion Simulator  
**LF01142U** Auscultation Trainer  
**LF01162U** Venatech IV Trainer  
**LF03000U** **CPARLENE**® Series  
**LF03601U** Adult Airway Management Trainer  
**LF03602U** Adult Airway Management on Manikin  
**LF03603U** Adult Airway Management Head Only  
**LF03609U** Child Airway Management Trainer  
**LF03610U** Child Airway Management Trainer Head Only  
**LF03611U** Child Defibrillation Chest Skin  
**LF03612U** Child IV Arm  
**LF03613U** Child Blood Pressure Arm  
**LF03614U** Child Intraosseous Infusion/Femoral Access Leg Only  
**LF03615U** Complete Child **CRiSis**™ Update Kit  
**LF03616U** Child **CRiSis**™ Manikin  
**LF03617U** Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor  
**LF03620U** PALS Update Kit  
**LF03621U** Infant Airway Management Trainer Head Only  
**LF03622U** Intraosseous Infusion Right Leg  
**LF03623U** Infant Airway Management Trainer  
**LF03626U** Child Femoral Access Injection Pad Replacement  
**LF03632U** Child Intraosseous Infusion/Femoral Access Leg on a Stand  
**LF03633U** Child Airway Management Trainer with Torso  
**LF03693U** **Basic Buddy** CPR Manikin  
**LF03699U** "Airway Larry" Airway Management Trainer  
**LF03720U** **Baby Buddy** Infant CPR Manikin  
**LF03953U** **CRiSis**™ Manikin  
**LF03955U** Deluxe **CRiSis**™ Manikin  
**LF04001U** **GERi**™ Nursing Manikin  
**LF04020U** **KERi**™ Nursing Manikin  
**LF04021U** **KERi**™ Basic Manikin  
**LF04022U** **KERi**™ Advanced Manikin  
**LF04030U** **GERi**™ Advanced Manikin  
**LF04040U** **GERi**™ Basic Manikin

*Nasco* Fort Atkinson

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
(920) 563-2446

[www.eNASCO.com](http://www.eNASCO.com) • E-Mail [lifeform@eNASCO.com](mailto:lifeform@eNASCO.com)