



**CHILD AIRWAY MANAGEMENT  
TRAINER HEAD FOR  
RESUSCI<sup>®</sup> JUNIOR\*  
LF03610  
INSTRUCTION MANUAL**

\*RESUSCI<sup>®</sup> JUNIOR IS A TRADEMARK OF THE LAERDAL CORPORATION.



**Life/form**® Products by NASCO


# NASCO *Life/form*® Child Airway Management Trainer Head for Resusci® Junior\*

## About the Simulator . . .

The *Life/form*® Child Airway Management Trainer Head is the most realistic simulator available for the training of intubation and resuscitation skills. NASCO has taken great care to create a trainer that is anatomically correct in respect to both size and detail.

Landmarks include: nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

The *Life/form*® Child Airway Management Trainer Head allows you to practice oral, digital, and nasal intubation, as well as CPR. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

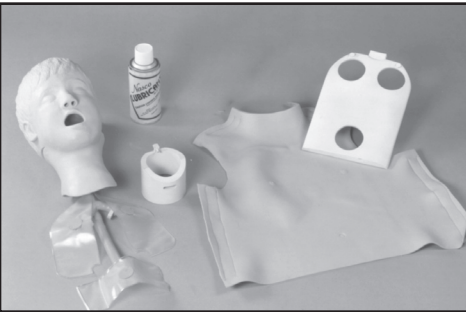


## List of Components:

1. Child Airway Management Trainer
2. Right and Left Lung Bags
3. Stomach Simulator
4. Chest Skin
5. Neck Adapting Ring
6. Chest Compression Plate

7. 4 oz. Can Lubricant
8. Three Nylon Hose Clamps

## Removing the Resusci® Junior\* Head:

1. Unbutton chest skin from sides and fold over face.
2. Unhook top of rib plate and lift out.

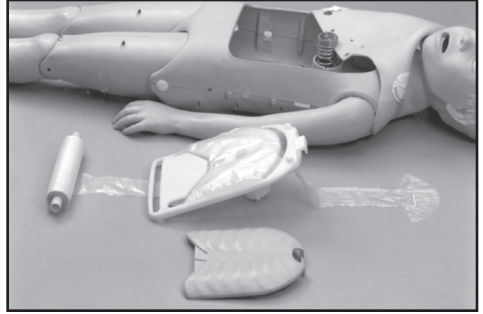

**Figure 1**


3. Remove the lung bag system from the Resusci® Junior\* body. Lift out plate. Remove roll of bags (Figure 1).
4. Detach the rubber tether strap from the pin at the base of the neck.



**Figure 2**

5. Turn the manikin on its side and loosen the four screws that hold the two halves of the upper body together. These are located across the back of the shoulders (Figure 2).


\*RESUSCI® JUNIOR IS A TRADEMARK OF THE LAERDAL CORPORATION.

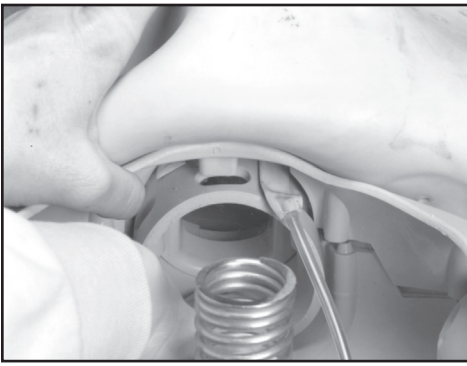

**Figure 3**


6. Lift the top half of the body slightly away from the bottom, releasing the tab that holds the neck in place (Figure 3).
7. Remove head and neck from body. Remove chest skin.

### Installing the *Life/form*<sup>®</sup> Child Airway Management Trainer Head:

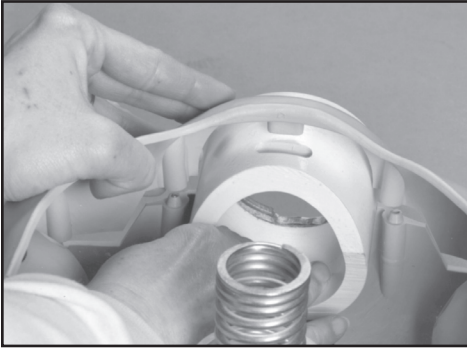


**Figure 4**

1. To install the neck adapter ring, separate the body halves slightly and insert adapter ring into body until the tab locks (Figure 4).
2. Attach the rubber tether strap to the peg on adapter ring.
3. Tighten the shoulder screws.

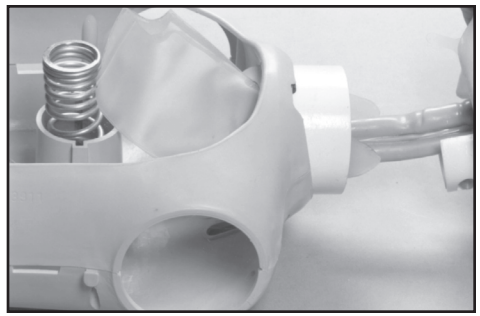

**Figure 5**

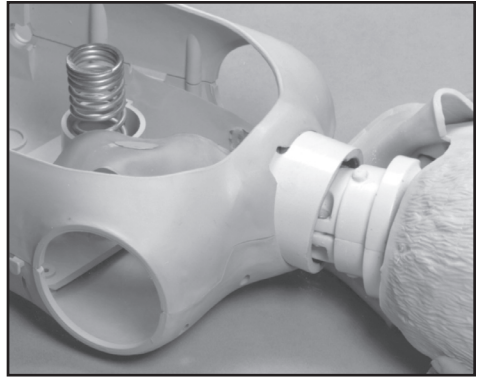

**Figure 6**

4. Carefully feed trachea, lungs, esophagus, and stomach through neck opening (Figure 5). Turn head and neck to side with the small key on the neck facing up. Insert into keyway of adapter ring and turn head face up (Figure 6).
5. Position the NASCO compression plate (Do not use the Resusci<sup>®</sup> Junior\* compression plate) over the large spring. Roll up lung and stomach bags and carefully feed each through the corresponding holes in the plate (Figure 7). Note that the bags should lie on top of the plate, with the connectors and hose clamps underneath (Figure 8).

\*RESUSCI<sup>®</sup> JUNIOR IS A TRADEMARK OF THE LAERDAL CORPORATION.

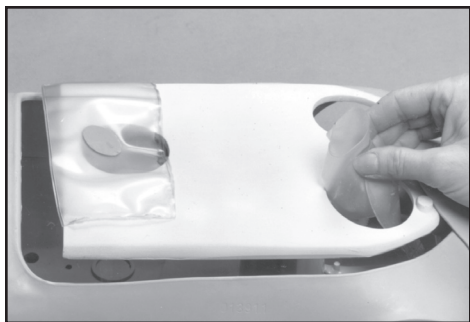


Figure 7



Figure 8

6. Hook rib plate onto chest compression plate, over bags.
7. Button the chest skin onto the body with the neck skin lying over the top. Your **Life/form**<sup>®</sup> Child Airway Management Trainer is now ready to use.

### Lubrication:

Before using the **Life/form**<sup>®</sup> Child Airway Management Trainer, lubricate both the oral cavity (Figure 9) and the endotracheal tube (Figure 10) with the NASCO lubricant provided. **NOTE:** NASCO recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Airway Management Trainer. The use of a silicone or other similar lubricant may seriously damage the simulator, thus voiding NASCO's warranty on the trainer.



Figure 9

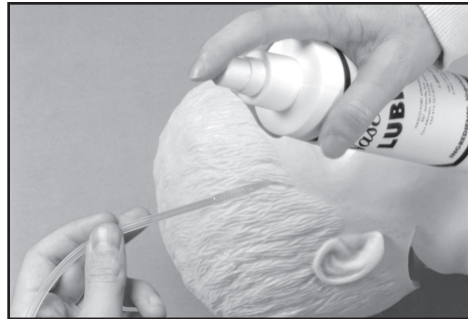


Figure 10

### Cleaning the Face:

Normal soil can be removed from the face with mild soapy water. **REN** cleaner (W09919U) will remove stubborn stains. Simply apply **REN** to soiled area and wipe clean with a soft cloth or paper towel. **NOTE:** Avoid using cleaner around mouth area if students will be applying direct mouth-to-mouth resuscitation techniques, as the cleaner may be toxic if ingested. **NEVER** place the trainer on any kind of printed paper or plastic. These materials, as well as ballpoint pens, will transfer indelible stains. Do not use any cosmetics.

### Available Supplies:

- LF03608U** NASCO Lubricant
- LF03627U** Replacement Lungs
- LF03628U** Replacement Stomach
- LF03631U** Replacement Headskin with Airway
- W09919U** **REN** Cleaner

**Additional Add-On  
Components for Resusci®  
Junior\*:**

- LF03611U** Defibrillation Chest  
Skin
- LF03612U** Left Arm — IV Training
- LF03613U** Right Arm — Blood  
Pressure Monitoring
- LF03614U** Right Leg —  
Intraosseous  
Infusion/Femoral  
Access

\*RESUSCI® JUNIOR IS A TRADEMARK OF THE LAERDAL CORPORATION.

## Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)  
**LF00856U** Female Catheterization  
**LF00901U** Prostate Examination  
**LF00906U** Ostomy Care  
**LF00929U** Surgical Bandaging  
**LF00957U** Enema Administration  
**LF00958U** Pediatric Injectable Arm  
**LF00961U** Intramuscular Injection  
**LF00984U** Breast Examination  
**LF00995U** Arterial Puncture Arm  
**LF00997U** Adult Injectable Arm (Black)  
**LF00999U** Pediatric Injectable Head  
**LF01008U** Intradermal Injection Arm  
**LF01012U** Heart Catheterization (TPN)  
**LF01019U** Ear Examination  
**LF01020U** Supplementary Ear Set  
**LF01025U** Male Cath-Ed I  
**LF01026U** Female Cath-Ed II  
**LF01027U** Peritoneal Dialysis  
**LF01028U** Suture Practice Arm  
**LF01036U** Spinal Injection  
**LF01037U** Hemodialysis Arm  
**LF01062U** Pelvic, Normal & Abnormal  
**LF01063U** Stump Bandaging, Upper  
**LF01064U** Stump Bandaging, Lower  
**LF01069U** Cervical Effacement  
**LF01070U** Birthing Station  
**LF01082U** Cricothyrotomy  
**LF01083U** Tracheostomy Care  
**LF01084U** Sigmoidoscopic Examination  
**LF01087U** Central Venous Cannulation  
**LF01095U** Blood Pressure Arm  
**LF01108U** Intraosseous Infusion Simulator  
**LF01121U** Advanced IV Arm  
**LF01139U** Advanced IV Hand  
**LF01142U** Auscultation Trainer  
**LF01162U** Venatech IV Trainer  
**LF03000U** **CPARLENE®** Series  
**LF03601U** Adult Airway Management Trainer  
**LF03602U** Adult Airway Management on Manikin  
**LF03603U** Adult Airway Management Head Only  
**LF03609U** Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only  
**LF03611U** Child Defibrillation Chest Skin  
**LF03612U** Child IV Arm  
**LF03613U** Child Blood Pressure Arm  
**LF03614U** Child Intraosseous Infusion/Femoral Access Leg Only  
**LF03615U** Complete Child **CRiSis™** Update Kit  
**LF03616U** Child **CRiSis™** Manikin  
**LF03617U** Deluxe Child **CRiSis™** Manikin with Arrhythmia Tutor  
**LF03620U** PALS Update Kit  
**LF03621U** Infant Airway Management Trainer Head Only  
**LF03622U** Intraosseous Infusion Right Leg  
**LF03623U** Infant Airway Management Trainer  
**LF03626U** Child Femoral Access Injection Pad Replacement  
**LF03632U** Child Intraosseous Infusion/Femoral Access Leg on a Stand  
**LF03633U** Child Airway Management Trainer with Torso  
**LF03693U** **Basic Buddy** CPR Manikin  
**LF03699U** "Airway Larry" Airway Management Trainer  
**LF03720U** **Baby Buddy** Infant CPR Manikin  
**LF03953U** **CRiSis™** Manikin  
**LF03955U** Deluxe **CRiSis™** Manikin  
**LF03965U** Deluxe "Plus" **CRiSis™** Manikin  
**LF04001U** **GERi™** Nursing Manikin  
**LF04020U** **KERi™** Nursing Manikin  
**LF04021U** **KERi™** Basic Manikin  
**LF04022U** **KERi™** Advanced Manikin  
**LF04030U** **GERi™** Advanced Manikin  
**LF04040U** **GERi™** Basic Manikin  
**LF06001U** **CPR Prompi™** Adult/Child Manikin  
**LF06012U** **CPR Prompi™** Infant Manikin  
**LF06200U** **CPR Prompi™** Keychain Rescue Aid  
**LF06200U** **CPR Prompi™** Rescue and Practice Aid

*Nasco* Fort Atkinson

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com