

Nasco
Life/form®

BASIC BUDDY™
CPR MANIKIN
LF03693U
INSTRUCTION MANUAL



Life/form® Products by NASCO

About the Simulator . . .

The **Life/form**® Basic Buddy™ CPR Manikin is the most realistic, economical manikin available for the training of CPR skills. Features include: sanitary airway system, head tilt, realistic chest rise, child and adult compression resistance, palpable landmarks, and a lightweight and compact design.

List of Components:

1. Base
2. Chest
3. Head
4. Aluminum Screw Post
5. Adult Compression Block
6. Package of Ten Lung Bags
7. Lung Installation Tool

See diagram for part reference.

Set Up:

Your Basic Buddy™ manikin is shipped disassembled. Simply follow these easy steps to assemble your manikin.

Attaching the Cover to the Base:

Place the chest on top of the base and lock it into place at the three points indicated in Figure 1.


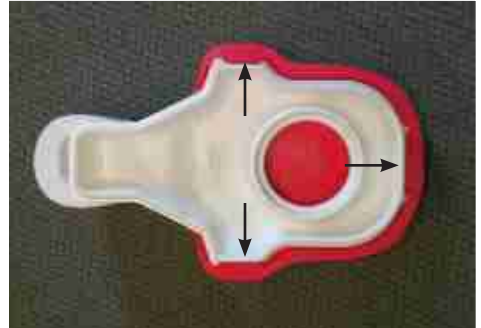


Figure 1

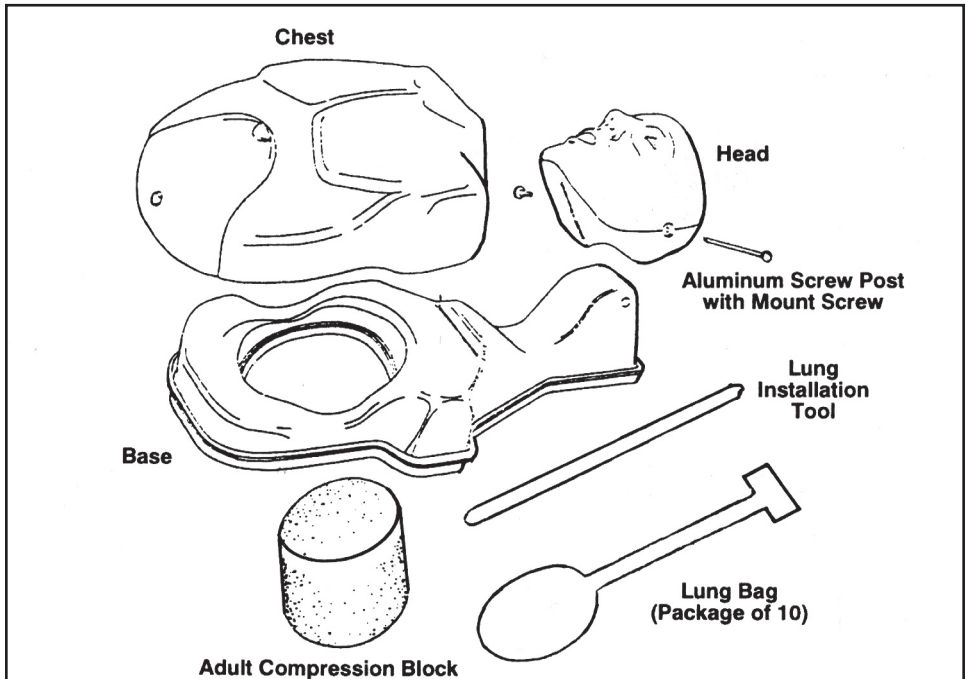

Attaching the Head to the Base:

Place the head over the neck area of the base and insert the aluminum screw post through the corresponding holes in the head and the base (Figure 2). Now insert the mounting screw into the end of the aluminum screw post and tighten (Figure 3).



Figure 2



Figure 3



Figure 4

Installing the Lung Bag:

Insert the lung installation tool into the lung bag (Figure 4). Then insert the bag and tool together into the manikin's mouth while feeding it through the head down to the chest (Figure 5). Continue feeding the tool and the lung bag into the chest cavity until the tool stops against the inside of the chest cavity (Figure 6). Remove the lung tool. Make sure the airway lies directly beneath the airway cutoff ridge of the head. Fold the barrier flaps of the lung bag over the face (Figure 7).

Your Basic Buddy™ is now set up for child CPR simulation.



Figure 5



Figure 6



Figure 7

Adult CPR Set Up:

To set up your Basic Buddy™ for adult CPR simulation, simply roll the manikin on its side and insert the adult compression block (Figure 8).



Figure 8

Removing the adult compression block will change your manikin back to the child compression resistance.

Maintenance:

The old lung bag should be removed whenever a new student will be working on the Basic Buddy™ manikin. Gently pull the lung out of the chest and mouth through the neck opening to avoid contaminating the manikin. Insert a new lung for each new student.



Figure 9

If you have purchased either the LF03694U Basic Buddy™ 5-Pack or the LF03695U Basic Buddy™ 10-Pack, refer to Figure 9 for the proper packing arrangement of five fully assembled manikins into the LF03697U nylon carrying bag.

Supplies/Replacement Parts for the Basic Buddy™ CPR Manikin:

- LF03693U** Basic Buddy™ CPR Manikin
- LF03694U** Basic Buddy™ CPR Manikins — 5 pack
- LF03695U** Basic Buddy™ CPR Manikins — 10 pack
- LF03696U** Basic Buddy™ Airways — 100 pack
- LF03697U** Nylon carrying bag (holds 5 Basic Buddy™ CPR Manikins)
- LF03698U** Basic Buddy™ Lung Installation Tool

Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompt**™ Adult/Child Manikin
LF06012U **CPR Prompt**™ Infant Manikin
LF06200U **CPR Prompt**™ Keychain Rescue Aid
LF06200U **CPR Prompt**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com