



---

**BABY BUDDY™ INFANT  
CPR MANIKIN  
LF3720U  
INSTRUCTION MANUAL**

---



**DO NOT STORE IN TEMPERATURES ABOVE 120°**

***Life/form***® Products by NASCO

## About the Simulator

The **Life/form**® Baby Buddy™ infant CPR manikin is the most realistic, economical manikin available for the training of infant CPR skills. Features include lifelike body and head, sanitary airway system, head tilt, realistic chest rise, palpable landmarks, and an easy-to-use, lightweight design.

## List of Components

Torso  
Chest  
Head  
Aluminum Screw Post  
2 Nylon Washers  
Package of 10 Lung Bags  
Lung Installation Tool

## Set Up

Your Baby Buddy™ manikin is shipped disassembled. Simply follow these easy steps to assemble your manikin.



**Figure 1**


### Attaching the Head

Position the head over the neck stub of the torso. Insert the screw post through one of the nylon washers and the corresponding holes in the head and neck (Figure 1). You may find this easier to do if you turn the manikin over.



**Figure 2**

Securely screw the end onto the screw post over the second washer (Figure 2).

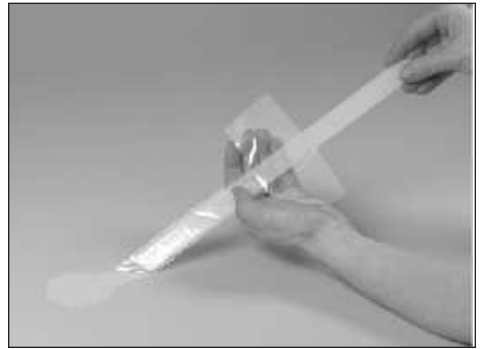

**Figure 3**

### Installing the Lung Bag

Insert the lung installation tool into the lung bag (Figure 3). Fold the round sides of the lung around the tool. Feed the lung into the mouth and through the hole in the back of the head (Figure 4).



**Figure 4**



**Figure 5**

With the head tilted back, grasp the bag with your other hand (Figure 5) and remove the tool. Gently pull the rest of the bag through until the barrier flaps reach the mouth. Lay the lung over the corresponding area of the torso. Make sure that the bag is not twisted or pinched, but is open for its entire length. Open the barrier flaps to protect the face (Figure 6).



**Figure 6**

Flex the foam chest piece slightly to snap it onto the torso over the lung (Figure 7). Your Baby Buddy™ is now ready to use.



**Figure 7**

## Maintenance

Used lung bags should be removed and discarded after each use. Always use a new lung bag for each student. The manikin's face may be cleaned with a mild detergent and water if it becomes soiled. Do not attempt to launder the foam pieces or immerse them in water.

## Supplies and Replacement Parts

- LF03720U** Baby Buddy™ Infant CPR Manikin
- LF03721U** Baby Buddy™ 5-Pack (Includes 1 Nylon Carry-All)
- LF03722U** Baby Buddy™ 10-Pack (Includes 2 Nylon Carry-Alls)
- LF03723U** Baby Buddy™ Lungs 100-Pack
- LF03724U** Nylon Carry-All (Holds 5 Baby Buddy™ Manikins)
- LF03725U** Baby Buddy™ Lung Installation Tool

## Other Available *Life/form*® Simulators

<b>LF00698U</b>	Adult Injectable Arm (White)	<b>LF03610U</b>	Child Airway Management Trainer Head Only
<b>LF00856U</b>	Female Catheterization	<b>LF03611U</b>	Child Defibrillation Chest Skin
<b>LF00901U</b>	Prostate Examination	<b>LF03612U</b>	Child IV Arm
<b>LF00906U</b>	Ostomy Care	<b>LF03613U</b>	Child Blood Pressure Arm
<b>LF00929U</b>	Surgical Bandaging	<b>LF03614U</b>	Child Intraosseous Infusion/ Femoral Access Leg Only
<b>LF00957U</b>	Enema Administration	<b>LF03615U</b>	Complete Child <b>CRiSis</b> ™ Update Kit
<b>LF00958U</b>	Pediatric Injectable Arm	<b>LF03616U</b>	Child <b>CRiSis</b> ™ Manikin
<b>LF00961U</b>	Intramuscular Injection	<b>LF03617U</b>	Deluxe Child <b>CRiSis</b> ™ Manikin with Arrhythmia Tutor
<b>LF00984U</b>	Breast Examination	<b>LF03620U</b>	PALS Update Kit
<b>LF00995U</b>	Arterial Puncture Arm	<b>LF03621U</b>	Infant Airway Management Trainer Head Only
<b>LF00997U</b>	Adult Injectable Arm (Black)	<b>LF03622U</b>	Intraosseous Infusion Right Leg
<b>LF00999U</b>	Pediatric Injectable Head	<b>LF03623U</b>	Infant Airway Management Trainer
<b>LF01008U</b>	Intradermal Injection Arm	<b>LF03626U</b>	Child Femoral Access Injection Pad Replacement
<b>LF01012U</b>	Heart Catheterization (TPN)	<b>LF03627U</b>	Child <b>CRiSis</b> ™ Replacement Lung Set
<b>LF01019U</b>	Ear Examination	<b>LF03628U</b>	Child <b>CRiSis</b> ™ Replacement Stomach
<b>LF01020U</b>	Supplementary Ear Set	<b>LF03629U</b>	Child IV Arm Replacement Skin and Veins
<b>LF01025U</b>	Male Cath-Ed I	<b>LF03630U</b>	Child Arrhythmia Tutor
<b>LF01026U</b>	Female Cath-Ed II	<b>LF03631U</b>	Child Airway Management Trainer Replacement
<b>LF01027U</b>	Peritoneal Dialysis	<b>LF03632U</b>	Headskin Child Intraosseous Infusion/ Femoral Access Leg on a Stand
<b>LF01028U</b>	Suture Practice Arm	<b>LF03633U</b>	Child Airway Management Trainer with Torso
<b>LF01036U</b>	Spinal Injection	<b>LF03650U</b>	<b>CRiSis</b> ™ Manikin
<b>LF01053U</b>	Cross-Sectional Anatomy, Torso, Head	<b>LF03651U</b>	ALS Update Kit
<b>LF01054U</b>	Cross-Sectional Anatomy, Head	<b>LF03675U</b>	BLS Manikin — 10 Pack
<b>LF01062U</b>	Pelvic, Normal & Abnormal	<b>LF03676U</b>	BLS Manikin — 5 Pack
<b>LF01063U</b>	Stump Bandaging, Upper	<b>LF03677U</b>	BLS Manikin — Single
<b>LF01064U</b>	Stump Bandaging, Lower	<b>LF04001U</b>	<b>GERI</b> ™ Nursing Manikin
<b>LF01068U</b>	Brachial Plexus	<b>LF04020U</b>	<b>KERI</b> ™ Nursing Manikin
<b>LF01069U</b>	Cervical Effacement	<b>LF04021U</b>	<b>KERI</b> ™ Basic Manikin
<b>LF01070U</b>	Birth Station	<b>LF04022U</b>	<b>KERI</b> ™ Advanced Manikin
<b>LF01082U</b>	Cricothyrotomy	<b>LF04030U</b>	<b>GERI</b> ™ Advanced Manikin
<b>LF01083U</b>	Tracheostomy Care	<b>LF04040U</b>	<b>GERI</b> ™ Basic Manikin
<b>LF01084U</b>	Sigmoidoscopic Examination		
<b>LF01087U</b>	Central Venous Cannulation		
<b>LF01094U</b>	Cross Sectional Anatomy — Laminated		
<b>LF01095U</b>	Blood Pressure Arm		
<b>LF01108U</b>	Intraosseous Infusion Simulator		
<b>LF03000U</b>	<b>CPARLENE</b> ® Series		
<b>LF03601U</b>	Adult Airway Management Trainer		
<b>LF03602U</b>	Adult Airway Management on Manikin		
<b>LF03603U</b>	Adult Airway Management Head Only		
<b>LF03609U</b>	Child Airway Management Trainer		

*Nasco* Fort Atkinson

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
(920) 563-2446

[www.eNASCO.com](http://www.eNASCO.com) • E-Mail [lifeform@eNASCO.com](mailto:lifeform@eNASCO.com)